

St. David's Med/Surg Units Only


WelchAllyn®

Partners in Care™ Services


FOR END USERS

Capturing and Sending Vital Signs with Connex® Spot Monitor:

SPOT Profile

- 1) **Verify connectivity** via Wireless symbol in top right corner of screen.
- 2) Touch **Clinician Symbol**  in top left corner of screen.
- 3) Touch Clinician ID Box, enter ID, touch OK. Touch password box, enter password, touch OK. Press Sign-In.
- 4) From the Home screen, scan **Patient ID**. Visually verify correct patient identification under Patient Name and Patient ID at the top of the screen.
- 5) Capture vitals using proper technique.
- 6) Tap on the **Modifiers Box** to enter NIBP, SpO2, Temp, or any additional modifiers as desired.
- 7) Within the Modifiers section press "Additional Parameters" to add additional items as required (Resp/ LOC) for EWS. Press next to calculate **MEWs**. Press OK to return to home screen.
- 8) Remove SpO₂ sensor from patient's extremity (*prior to touching SAVE*).
- 9) Touch Save (was the next button); visually confirm "Save Successful/Send Successful" messages.

 Wireless symbol

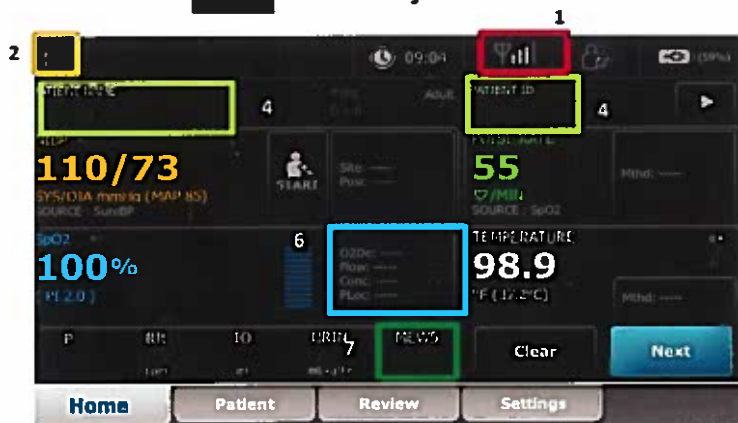

MC 13668 10/2016